SE

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EN


The Eve cup is your new menstrual cup: it is innovative, comfortable, environmentally-friendly and the best solution for your intimate hygiene. Made of 100% Platinum-silicone, it optimizes the body's tolerance and can be used for up to 12 hours.

Why use the Eve cup?

- No allergies.
- It does not cause any vaginal thrush, etc, or urinary inflammation.
- It has no negative effect on vaginal health.
- It is ideal for application in sports, comfortable at night and during the day and great for travelling.

How do I insert my Eve cup?

Before you use it for the first time, we recommend you sterilize the cup (boiling it in water for a few minutes or using a sterilization solution). Always insert it with clean hands, sitting, squatting, kneeling or standing, in a relaxed position. You are free to choose what is best for you. To make this easier, you can moisten the Eve cup with water or use a water based lubricant.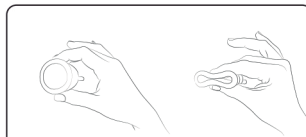

①


1: Press the cup in your hand

②

2: Fold the cup in half lengthwise


3: Insert the cup in the vagina obliquely, toward the back, bearing in mind that its position has to be lower than in the case of tampons.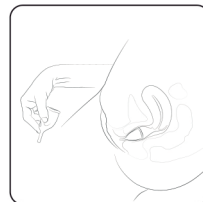

③

How do I remove my Eve cup?

Make sure your hands are thoroughly clean! In the position of your choice, relax and contract your vaginal muscles and ease the cup slowly outwards from your vagina. Use a finger to check that the rim are well-unstuck from the vagina walls. Empty the contents into the toilet. Wash the cup with water and neutral soap, dry with clean paper, and place it in its pouch.

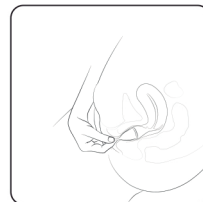How do I use my Eve cup?

Remove and empty the cup every 12 hours, at the most! Adapt the length of the stem to your needs, trimming off a small amount, if required.

Important warning!

Do not use the Eve cup for over 12 hours at the maximum! Clean the cup using only neutral soap! Only use water based lubricant only! Eve cup is not a contraceptive and doesn't protect against venereal diseases! In case of any vaginal symptoms, contact your gynecologist

FEMINTIMATE

ÈVE CUP

100% PLATINUM SILICON
PHTHALATE FREE

Phthalate Free

CNEX AIE S.L. MENDEZ NUÑEZ, 49
MATAFÓ 08032 BARCELONA, SPAIN
MADE IN PRC

DISTRIBUERAD AV:
JEWÄ SWEDEN AB
WWW.JEWÄSWEDEN.SE

FEMINTIMATE

ÈVE CUP

100% PLATINUM SILICON
PHTHALATE FREE

Phthalate Free

CNEX AIE S.L. MENDEZ NUÑEZ, 49
MATAFÓ 08032 BARCELONA, SPAIN
MADE IN PRC

DISTRIBUERAD AV:
JEWÄ SWEDEN AB
WWW.JEWÄSWEDEN.SE