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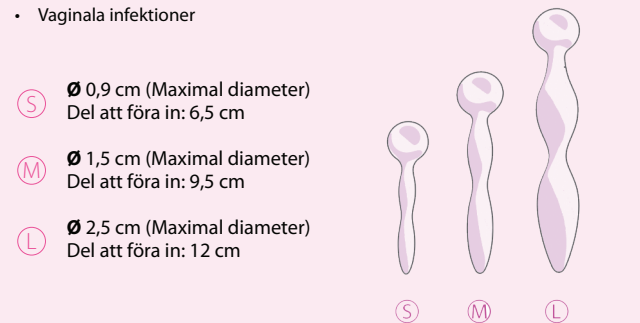
- Graviditet
- Menstruation
- Vaginala infektioner

Package content:
3 silicone items.

Manufacturer:
EastGlory International Hk Co., Ltd.
Foreign Industry Zone,
DongChen Area of DongGuan City,
China

Authorized Representative:
CNEX AIE, SL
C/Mendez Nuñez,49
08032 Mataró
BARCELONA-SPAIN
Tel:+ 34 933 806 526
Fax:+ 34 933 806 528

Rev. 3
Date 18/10/2013

Distribuerad av:
Jewa Sweden AB
www.jewasweden.se

INTIMRELAX is indicated for treating atrophic vaginitis or vaginal atrophy and, in general, vaginal contraction, burning sensation and penetration difficulty.

What is atrophy vaginitis?

There are diverse motives and they can originate from small sequelae of infections (inflammation of the vagina), atypical morphology (excessively narrow vaginal duct), congenital malformations (even slight ones), or simply from the apprehension of penetration.

This disorder is often accompanied by the reduction of oestrogen levels, resulting as a consequence in:

- A slimming of the vaginal walls and a loss of the mucosal folds of the latter.
- A decrease in the lubrication and elasticity of the vagina.
- Changes in the vaginal mucosa which becomes thinner and drier.
- Changes in the balance of the flora and in the pH (the degree of acidity)

The symptoms are:

- Dyspareunia (painful intercourse) stinging or burning sensation in the vaginal area. Small vaginal blood losses, especially after intercourse. It is often associated with a loss of sexual appetite.
- Dysuria: stinging or burning sensation when urinating.
- Vaginal prolapse: the vaginal walls drop in such a way that the vagina loses its normal position.

Infections, ulcers or injuries can also occur more easily in the vaginal wall.

Who does atrophic vaginitis affect?

Women after menopause. In fact, between 40% and 60% of women suffer from urogenital symptoms related in this stage of life. Younger women (it affects 10-15% of women of childbearing age), especially after giving birth or during breastfeeding. 5% of women who go to a gynaecology consultation refer to symptoms related to the vaginal atrophy.

Solution:

INTIMRELAX allows to do exercise and to progressively dilate the vaginal duct. It allows at the same time, with small exercises, the progressive improvement of the symptoms derived from atrophic vaginitis. It is made up of three elements with a perfectly ergonomic form and different sizes in order to gradually solve this disorder which has adverse effects on sexuality and psychological health in general.

INTIMRELAX should be used in conjunction with a water-based lubricant in order to make the exercises easier and at the same time lubricate and hydrate the vaginal mucosa.

Vaseline, which can damage the latex of condoms or diaphragms (due to risk of genital infection), or other lubricants which are oil based are completely contraindicated.

How is INTIMRELAX used?

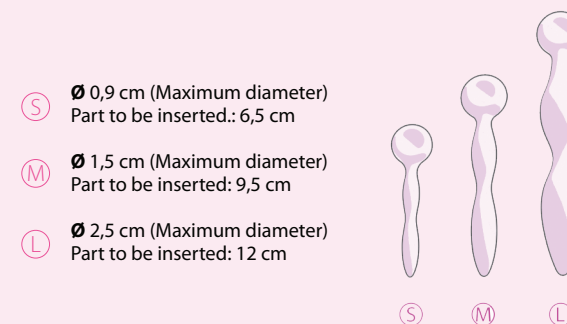
First of all, a small amount of water based lubricant should be applied to the entrance of the vagina (approximately a chestnut-sized amount) and it should be gently massaged for 2-3 minutes.

After having done so, the next step is to take the smallest component of INTIMRELAX. The same amount of INTIMRELAX-LUB lubricant should be likewise applied to the tip. Straightaway, pressure should be applied to insert it in the vagina in a gentle and progressive way. If dryness is perceived, more lubricant should be added.

This exercise can be done for 10 to 15 minutes with small movements in and out. It is important to bear in mind that the contraction of the vaginal muscles is adverse and if it occurs the exercise should be stopped. The objective is to relax the vaginal area as well as alleviating the discomfort caused by atrophic vaginitis. Therefore the exercises should be carried out in a comfortable place.

One session a day is enough.

When the smallest component can comfortably be inserted without discomfort as part of the exercises, they should be continued with a bigger component.

Package content:
3 silicone items.

Manufacturer:
EastGlory International Hk Co., Ltd.
Foreign Industry Zone,
DongChen Area of DongGuan City,
China

Authorized Representative:
CNEX AIE, SL
C/Mendez Nuñez,49
08032 Mataró
BARCELONA-SPAIN
Tel:+ 34 933 806 526
Fax:+ 34 933 806 528

Rev. 3
Date 18/10/2013

Distribuerad av:
Jewa Sweden AB
www.jewasweden.se